

## Customer guidance:

### Cleaning Stryker's Emergency Care products

**Many customers have reached out with questions regarding the type of cleaning agents that can be used on their Stryker devices.**

The following guidance can be used for cleaning the products listed below. This information can be referenced in the Operations Manual (OM)/Operating Instructions (OI)/Instructions for Use (IFU) for each product. The OM/OI/IFU remains the sole source for product information.

In addition to the cleaning agents listed below from the Stryker OM/OI/IFU, the Environmental Protection Agency (EPA) in the US has released a list of disinfectants suitable for COVID-19, some of which may be used on Stryker products, providing that the OM/OI/IFU do not say otherwise. The list is updated regularly and is available here: <https://www.epa.gov/pesticide-registration/list-n-disinfectants-use-against-sars-cov-2>. If a product or cleaning method is not listed in the applicable OM/OI/IFU, Stryker has not tested it and its use may damage the product.

Product(s)	Suggested cleaners
LIFEPAK® monitor/defibrillators LIFEPAK AEDs	<ul style="list-style-type: none"> <li>Quaternary ammonium compounds</li> <li>Isopropyl alcohol</li> <li>Peracetic (peroxide) acid solutions</li> </ul> <p><b>DO NOT</b> clean any part of this device or its accessories with bleach, bleach dilution, or phenolic compounds. Do not use abrasive or flammable cleaning agents.</p>
HeartSine® samaritan® PAD	<ul style="list-style-type: none"> <li>Soapy water</li> <li>Isopropyl alcohol (70% solution)</li> </ul> <p><b>DO NOT</b> clean the samaritan PAD with abrasive materials, cleaners or solvents.</p>
LUCAS® chest compression system	<ul style="list-style-type: none"> <li>70% isopropyl alcohol solution</li> <li>45% isopropyl alcohol solution with added detergent</li> <li>Quaternary ammonium compound</li> <li>10% bleach</li> <li>Peracetic (peroxide) acid solutions</li> </ul>
Power-PRO™ powered ambulance cot Performance-PRO™ manual ambulance cot	<ul style="list-style-type: none"> <li>Quaternary cleaners (active ingredient – ammonium chloride) that contain less than 3% glycol ether</li> <li>Phenolic cleaners (active ingredient – o-phenylphenol)</li> <li>Chlorinated bleach solution (5.25% – less than 1 part bleach to 100 parts water)</li> <li>≤ 21% isopropanol alcohol</li> </ul> <p><b>DO NOT USE</b> Virex® TB or Iodophor type disinfectants</p>



MX-PRO® R3 manual ambulance cot MX-PRO® bariatric transport cot Stair-PRO® stair chair	<p>Unless otherwise noted below, use concentrations recommended by the manufacturer:</p> <ul style="list-style-type: none"> <li>• Quaternary cleaners (active ingredient – ammonium chloride)</li> <li>• Phenolic cleaners (active ingredient - o-phenylphenol)</li> <li>• Chlorinated bleach solution (5.25% - less than 1 part bleach to 100 parts water)</li> </ul> <p><b><u>DO NOT USE</u></b> Virex® TB or Iodophor type disinfectants</p>
Altrix® temperature management system	Consult the product's Operations Manual and Disinfection Guide at <a href="http://techweb.stryker.com/Temp_Management/index.html">http://techweb.stryker.com/Temp_Management/index.html</a>
McGRATH® MAC EMS video laryngoscope	Consult the product's Instructions for Use directly at <a href="https://www.strykeremergencycare.com/disclosure-and-safety-information/">https://www.strykeremergencycare.com/disclosure-and-safety-information/</a>

If you have any questions about this matter, contact your Stryker representative or visit our website at [stryker.com](http://stryker.com)

## Resources

Links to Operating Instructions/Instructions for Use for all products:

- LIFEPAK Defibrillators, AEDs, CPR devices, video laryngoscope and accessories: <https://www.strykeremergencycare.com/disclosure-and-safety-information/>
- HeartSine samaritan PADs: <https://heartsine.com/support/product-manuals/>
- Ambulance cots and accessories: <https://techweb.stryker.com/EMS/index.html>
- Temperature management system: [http://techweb.stryker.com/Temp\\_Management/index.html](http://techweb.stryker.com/Temp_Management/index.html)

List N: Products with Emerging Viral Pathogens AND Human Coronavirus claims for use against SARS-CoV-2: <https://www.epa.gov/pesticide-registration/list-n-disinfectants-use-against-sars-cov-2>

Copyright © 2020 Stryker. 03/2020. GDR 3343728\_A

Stryker or its affiliated entities own, use, or have applied for the following trademarks or service marks: Altrix, HeartSine, LIFEPAK, LUCAS, MX-PRO, Performance-PRO, Power-PRO, Stair-PRO, Stryker. All other trademarks are trademarks of their respective owners or holders.

The absence of a product, feature, or service name, or logo from this list does not constitute a waiver of Stryker's trademark or other intellectual property rights concerning that name or logo.



**Emergency Care**

Fuego B.V. | Morsestraat 29, 4004 JP Tiel | +31 344 630 782 | [info@fuego-bhv.nl](mailto:info@fuego-bhv.nl) | [fuego-bhv.nl](http://fuego-bhv.nl)