

Cleaning and Disinfection instructions

for all types of Rotaid AED Cabinets

Cleaning

The cabinet may be cleaned by using a clean soft sponge and washing with lukewarm water containing a mild soap or a slightly acidic, neutral or slightly alkaline detergent. Then rinse thoroughly with clean water and dry with chamois leather or a moist sponge.

Suitable cleaning solvents at room temperature are water, white spirit, hexane, heptane, ethyl alcohol, butyl alcohol, isopropyl alcohol (also known as IPA or Isopropanol).

Disinfection

Disinfection of the surface can be performed using > 60% isopropyl alcohol (IPA), ethanol, or any other antibacterial or antiviral cleaning agent, sprayed or applied on a soft cloth, preferably followed by a thorough wash with clean water and rinse as described above.

AED

For cleaning of the defibrillator and its accessories, please contact your supplier and / or AED manufacturer.

Support

For any other questions and or support regarding the above, please contact us at:



Fuego B.V.

Morsestraat 29, 4004 JP Tiel | +31 344 630 782 | info@fuego-bhv.nl



StaySafe